

Friday, May 9, 2014

## Milkies Milk Saver Review And Giveaway

When I was pregnant with Bunny I saw Milkies Milk Saver in the store. I thought it looked weird and uncomfortable and I did not think it really had a valuable purpose. Because, really, nobody could possibly leak THAT much! Oh what little I knew!


Once Bunny was born and we started nursing, one side per feeding, I realized how wrong I had been! I ordered Milkies Milk Saver immediately! They couldn't get to me fast enough! When my Milkies Milk Saver arrived, I still thought it looked big and uncomfortable. I was pleasantly surprised when I used it! Milkies Milk Saver is VERY comfortable! I thought for sure having something pushing on my engorged breast would feel very uncomfortable, but Milkies Milk Saver does not push on your breast. It sits gently in your bra, against your breast. If you wear a tighter bra, it will definitely push more against your breast, but in the very beginning, I was only wearing sleep bras.



I used Milkies Milk Saver when I would nurse Bunny, and when I would pump. In the very beginning I had a single pump that only pumped one side at a time. If I did not use the Milkies Milk Saver, I would soak through my nursing pads. I could collect about an ounce or so of milk from the side that was not being used! If you have supply issues or are having a hard time keeping up with your baby's intake, an ounce is a massive amount to waste in your nursing pad!



Using Milkies Milk Saver is very easy. You simply set Milkies Milk Saver in your bra with your nipple in the hole and nurse or pump on the other side! Milkies Milk Saver can comfortably hold 2 ounces of breast milk before you need to empty it. It is easy to switch from one side to the other side without spilling any milk. The only time I ever spilled any milk is when I forgot I had the Milkies Milk Saver on (it is that comfortable!) and I bent over. Out spilled all the milk! Oops! I have heard some women say they wear it all day long when they are out and about, but due to the spilling issue, I do not see that working very well.

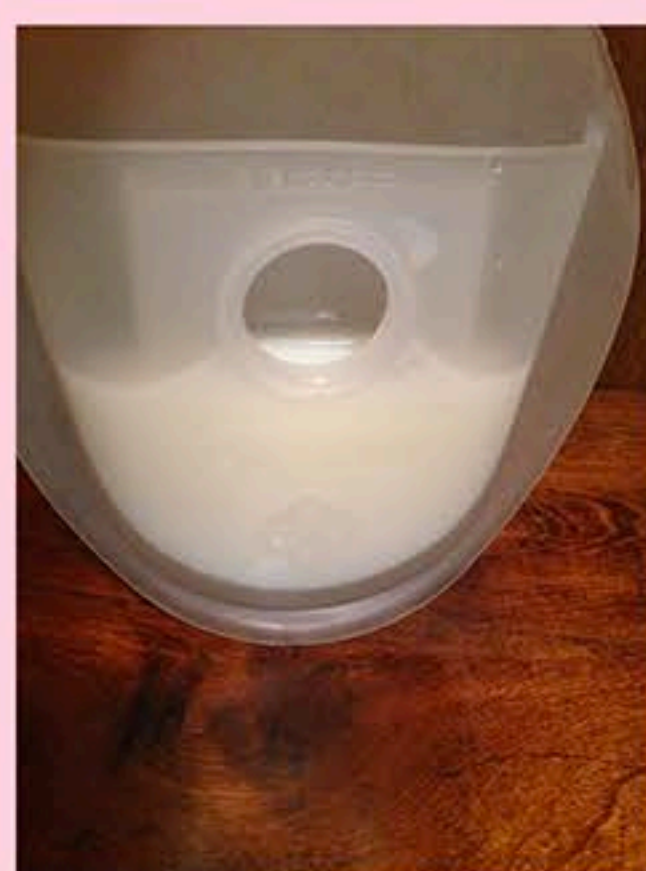

2 ounces of milk



2 ounces of milk

Milkies Milk Saver is very easy to clean. You can hand wash it with warm water and soap. It air dries fairly quickly, or you can use a towel to dry it off.

Milkies comes in a protective case that is great for storing your Milkies in when not in use. The case is a hard material to keep Milkies protected. The case also doubles as a stand. You can set Milkies in the case upright with milk inside. This is very helpful when you have a squirmy baby in your arms, when you need to put yourself back together after nursing, or when you need to grab a container to pour the milk into.



Milkies in the case



Milkies in the case

I absolutely LOVE this product! I used it multiple times a day for about a year. I am finally at the point where I do not leak every time I nurse. For those early days when I was leaking up a storm, this was a wonderful way to collect breast milk, and keep me from making a mess.



Milkies can sit in the case while you rearrange yourself, without spilling!

Search This Blog

Subscribe

Share It

[Share this on Facebook](#)

[Tweet this](#)

[View stats](#)

[\(NEW\) Appointment gadget >>](#)

